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STERILE HYPODERMIC SYRINGE FOR SINGLE USE (WITH NEEDLE)

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1. CHECK THE PACKAGING OF THE SYRINGE. WHETHER THE PACKAGE HAS BEEN OPENED OR DAMAGED. 2. CHECK THE COLOR OF THE NEEDLE, CHOOSE THE SIZE OR NEEDLE THAT IS SUITABLE FOR THE PURPOSE OF USE. 3. TEAR THE PACKAGE ALONG THE TEAR MARK ON THE PACKAGE AND REMOVE THE PRODUCT. 4. TURN AND TIGHTEN THE NEEDLE WITH THE TIP OF THE SYRINGE TO PREVENT LEAKAGE. 5. THEN REMOVE THE COVER. USE A NEEDLE TO SUCK UP THE REQUIRED AMOUNT OF MEDICATION IN THE MEDICAL VIAL. 6. USE THE TIP OF THE NEEDLE TO INJECT INTO THE SKIN AT AN ANGLE OF APPROXIMATELY 90 DEGREES. 7. USE THE THUMB AGAINST TO PUSH IT SLOWLY. 8. WHEN THE MEDICINE IS OVER, USE A COTTON PAD TO PRESS THE NEEDLE IN PLACE. WHILE WITHDRAWING THE NEEDLE QUICKLY 9. AFTER USE, DISCONNECT THE NEEDLE FROM THE SYRINGE AND STORE IT IN THE CONTAINER.

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• DISCARD AFTER SINGLE USE • DO NOT STORE AT EXTREME TEMPERATURES  $\geq 104.00^{\circ}\text{F}$  ( $40^{\circ}\text{C}$ ). • DO NOT USE IT IF THE PACKAGE IS OPENED OR DAMAGED • DO NOT USE AFTER EXPIRY DATE

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STERILE in unopened, Undamaged package, Non-pyrogenic, Do not use with paraldehyde.

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The Syringes are assembled by Barrel, Plunger, Piston without Hypodermic Needles. All parts and material for this product meet medical requirements.

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Sterile syringes are intended to inject drug for patients. And syringes are intended for use immediately after filling and are not intended to contain the medicament for extended periods of time.

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Should be stored in a dry place away from high temperature and humidity. This prevents syringes or pouches from deteriorating or losing their sterility.

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The liquid inside the syringe should be injected as soon as the liquid is filled. The liquid should not be stored in the syringe for a long time.

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